



Committed to Excellence
Technical & General Ltd

V1 04/2017

Code: XI DA1
Code: XI DA5

tgdaily Concentrated Aspirator Daily

For the daily cleaning and maintenance of Dental Aspirator Systems.

Application

- Highly effective cleaning and disinfecting product for all types of aspiration equipment.
- Can be used for effective cleaning and disinfection of spittoon bowl and amalgam separators.
- Can be used on wet line, dry line and semi wet/ dry line systems.
- Non foaming. Non-abrasive.
- Phenol free. Aldehyde free. Chlorine free.
- Colour coded.
- Easy to use and measure.

Effective Spectrum

- Effective against bacteria, fungi, viruses and diverse range of micro-organisms.

Instructions for use

Dispense 5% solution (50ml Auto-rings Weekly to 950ml warm water). Aspirate via the high and low volume suction tubes and also dispense into the spittoon bowl. Leave for a minimum of 10 minutes in order to activate the working properties and then rinse thoroughly with clean water.

Composition

Alkyl Dimethyl Amonium Chloride, Tetrasodium Ethylene-diamineTetraacetate, FattyAcid Amido Alkyl Betaine.



Available in 1 and 5 Litre containers.

Sales and Information

Technical & General Ltd., 2 Albion Place, London W6 0QT
Tel. 020 8834 7575 Email: sales@technical-general.com
www.dentistmarket.com www.tgdent.com



0005