

ETCHANT

The success of the composite and dentin/enamel bonding techniques depend on a correct tooth enamel etching.

Procedures:

Clean and dry the cavity and tooth.

Etch the enamel of the tooth with the etchant acid for 20 or 30 sec. Rinse and dry with clean air.

If the enamel does not have a chalk aspect repeat the process.

For Professional Dental use only



TECHNICAL & GENERAL Ltd. WARNINGS

Intention for use in etchant gel or liquid are use to etch the enamel of the teeth, creating a mechanical retention for all the resins and resins composite techniques.

Composition:
Phosphoric acid
Bidistilled water
Pigment
Gelable agents.

The etchant is not irritable to the skin. In case of contact with the eyes, wash immediately with abundant water for at least 15 minutes and if the irritation still persist consult a doctor. In case of contact with skin wash the area with soap and water. In case of inhalation clean the nose with water and seek medical assistance. Before the use of these products always use rubber dam.

Some patients can present allergy to the etchant or any of its components. In this case stop its use. Do not store at temperature above 24°C. Close the packages immediately after dispensing of the product.

Keep out of reach of children.
For professional dental use only.

Contraindication: The material must not be used when the application techniques described are not possible.

Warranty: Technical & General Ltd. Warrants the product against any fault of manufacture of the material. Technical & General Ltd., does not take responsibility for the misuse of the product out of the intention for use specified or out of the instructions for use, or out of the precautions, the warnings, and indicated contraindications. Being the only obligation is the replacement of the defective product.

Information: Post-production:

If the use of these products on a patient or any one else causes some anomaly, please send detailed report to Technical & General Ltd.



CONAMCO, S.A. de C.V.

José María Bustillos No. 28 Bis, 1er y 2do piso.
Colonia Algarín, Delegación Cuauhtémoc, CP.
Ciudad de México, 06880, Mexico



Ivonne Richaud.

Travessera de Gracia, 315, 2º-1ª 08025
Barcelona, Spain

NO REFRIGERATION NECESSARY