

LIGHT CURE ORTHODONTIC ADHESIVE SYSTEM

1. Perform a prophylaxis with a non-fluoride, oil-free paste. Isolate and dry each tooth to be sealed with oil-free air.
2. Apply the acid etching and allow the surface to etch for 30 seconds. Rinse well and dry with oil-free air. If the chalk look does not appear repeat the operation.
3. Place one drop of bonding resin over the tooth surface and apply the light from a source lamp for 15 seconds.
4. Place one drop of bonding resin over the bracket posterior face and apply the light from a source lamp for 15 seconds.
5. Apply a little portion of paste from the syringe over the bracket posterior face and place and position it over the tooth with a little pressure. Apply the light around the bracket and to the opposite side of the tooth for one minute.
6. After 5 minutes, you can activate the bracket.

For Professional Dental use only



TECHNICAL & GENERAL Ltd. WARNINGS

Intention for use in cementation of orthodontic bracket.

Composition:
Crystalline and fumed silicon dioxide
Bis-gma
Stabilizer
Photo initiators.

The light cure orthodontic adhesive system is not irritable to the skin. In case of contact with the eyes, wash immediately with abundant water for at least 15 minutes and if the irritation still persist consult a doctor. In case of contact with skin wash the area with soap and water. In case of inhalation clean the nose with water and seek medical assistance. Before the use of these products always use rubber dam.

Some patients can present allergy to the light cure orthodontic adhesive system or any of its components. In this case stop its use. Do not store at temperature above 24°C. Close the packages immediately after dispensing of the product.

Keep out of reach of children.
For professional dental use only.

Contraindication: The material must not be used when the application techniques described are not possible.

Warranty: Technical & General Ltd. Warrants the product against any fault of manufacture of the material. Technical & General Ltd., does not take responsibility for the misuse of the product out of the intention for use specified or out of the instructions for use, or out of the precautions, the warnings, and indicated contraindications. Being the only obligation is the replacement of the defective product.

Information: Post-production:
If the use of these products on a patient or any one else causes some anomaly, please send detailed report to Technical & General Ltd.



CONAMCO, S.A. de C.V.
José María Bustillos No. 28 Bis, 1er y 2do piso.
Colonia Algarín, Delegación Cuauhtémoc, CP.
Ciudad de México, 06880, Mexico



Ivonne Richaud.
Travessera de Gracia, 315, 2º-1ª 08025
Barcelona, Spain

NO REFRIGERATION NECESSARY