

tgsealer is a Zinc Oxide Eugenol Root Canal Sealer. The powder contains: Zinc Oxide, Thymol Iodide, Barium Sulphate and Excipients. The liquid is Eugenol and Excipients.

Instructions

- 1) Place one drop of the liquid on a sterile slab, with two spoons of powder.
- 2) For precise dosage and lab test, weigh 0.5 gm of powder for each 0.05 ml of liquid.
- 3) Spatulate one portion of the powder into the liquid until all particles of the powder is dissolved. Add more portions of powder to the mix, spatulating thoroughly until a very smooth creamy paste is obtained. Spatulating time ranges from 3 to 5 minutes.

The longer the spatulating time, the faster the setting time. Stop spatulating when the mixed cement is of such consistency that a thin unbroken thread of at least 1 inch length spatulation may be obtained when the flat surface of the spatula is slowly raised from the mixture on slab.

- 4) At relative humidity of 40% and room temperature of 22C the final setting time on the glass slab is between 8.5 and 10 hours. Inside the root canal the initial setting time is approximately 20 minutes, thus enabling the dentist to make a correction should it be necessary.



TECHNICAL & GENERAL Ltd. WARNINGS

Intention for use in sealant of root canal.

Composition:

Zinc oxide

Eugenol

Bismuthe subnitrate

Rosin lump.

The root canal cement is not irritable to the skin. In case of contact with the eyes, wash immediately with abundant water for at least 15 minutes and if the irritation still persist consult a doctor. In case of contact with skin wash the area with soap and water. In case of inhalation clean the nose with water and seek medical assistance. Before the use of these products always use rubber dam.

Some patients can present allergy to the root canal or any of its components. In this case stop its use. Do not store at temperature above 24°C. Close the packages immediately after dispensing of the product.

Keep out of reach of children.
For professional dental use only.

Contraindication: The material must not be used when the application techniques described are not possible.

Warranty: Technical & General Ltd. Warrants the product against any fault of manufacture of the material. Technical & General Ltd., does not take responsibility for the misuse of the product out of the intention for use specified or out of the instructions for use, or out of the precautions, the warnings, and indicated contraindications. Being the only obligation is the replacement of the defective product.

Information: Post-production:

If the use of these products on a patient or any one else causes some anomaly, please send detailed report to Technical & General Ltd.



CONAMCO, S.A. de C.V.
José Ma. Bustillos No. 28-Bis, 1er. y 2do. Piso,
Col. Algarín, Alcaldía Cuauhtémoc,
C.P. 06880, Ciudad de México, México.



Ivonne Richaud.
Calle Travessera de Gracia No. 315, 2°-1°
C.P. 08025, Barcelona, España.

NO REFRIGERATION NECESSARY